# scala®

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# Digital Wrist Blood Pressure Monitor SC 7100

CE 0197

- Instructions for use -

## Introduction

Thank you for deciding for the blood pressure monitor SC 7100 from SCALA. To be able to utilise all advantages of the blood pressure monitor to the fullest, you should carefully read these instructions before using the monitor. Keep the instruction leaflet in a safe place.

## Safety instructions

The blood pressure monitor SC 7100 enables you to monitor your blood pressure regularly. However, you should never change the treatment which your doctor has prescribed

**Self-measurement means monitoring, it does not imply diagnosis or treatment.**

Discuss unusual values with your doctor.

**Do not change the medication prescribed by your doctor.**

The heart rate is not a useful indicator of the frequency of a cardiac pacemaker. Patients with cardiac arrhythmia should use the blood pressure monitor only after consulting a doctor.

## Features of the SC 7100

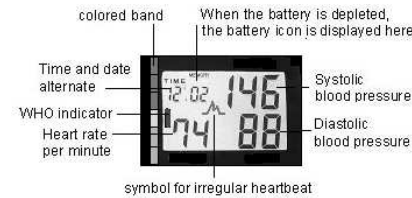
- The blood pressure monitor has fuzzy logic to identify the pumping pressure. If the pumping pressure set in the factory is not sufficient, the device detects this and continues pumping.
- The device has 2 selectable memories each with 50 memory locations.
- The date and time are displayed and stored together with every blood pressure value.
- WHO interpretation of blood pressure readings with coloured LED band (See description at page 7).
- If your heartbeat is not regular (irregular), the  icon appears in the LC display.
- If the battery icon is displayed, the batteries are discharged and the device cannot be used until new batteries are inserted.
- The handcuff is suitable for wrists from 135 to 195mm circumference.

## Preparing the monitor for measurement / changing batteries

Insert the two supplied batteries in the battery compartment at the side of the blood pressure monitor as shown. **Use only alkali batteries (LR03).**

If the batteries are depleted, the battery icon  is displayed in the top left corner of the LC display. The battery needs urgent replacement because you cannot use the monitor with depleted batteries.

## The LC display

## The buttons

Memory area select button

MEMORY button  
ON/OFF button

## Setting the clock

After you have inserted the two batteries, you should set the clock. It starts with year "2009", „1/ 1“ (for the 1st month / 1st day) and "12:00" o'clock after every battery change.

To set the clock, hold the MEMORY button pressed for about 5 seconds. The year 2009 blinks in the top left corner of the display.

*Set the year with the ON/OFF button.*

When the correct year is set, press the MEMORY button again. Now the 1 blinks in the top left corner of the LCD. It stands for the month.

*Set the month with the ON/OFF button.*

Another press of the MEMORY button

sets another 1 blinking. It stands for the first day in the month.

*Set the day with the ON/OFF button.*

Another press of the MEMORY button takes you to the clock setting – the hours blink. As before, *set the clock with the ON/OFF button.*

Pressing the MEMORY button advances to the minutes setting.

*Set the minutes with the ON/OFF button.*

A final press of the MEMORY button completes the setting of the date and time.

## How to obtain correct measuring results?

To obtain correct readings, measure the blood pressure always under the same conditions, at rest, not under the effect of alcohol, not during meals, not after heavy physical or mental stress. If possible, sit down at a table for the measurement.

Please note: The blood pressure monitor must be at heart level during the measurement, otherwise wrong readings are obtained.

## The measuring method

Put the cuff around your **left** wrist making sure that the device rests on the artery and the top edge of the device is about 1.5 cm away from the wrist joint.

The cuff should be applied with moderate tightness.

Now select "your" memory area with the select button, which you want to use and in which you want to store your blood pressure values by pressing the MEMORY button. (The BP monitor has 2 different memory areas each with 50 memory locations. The other memory area can be used by a family member or other person.)

After you press the button for selecting the memory area, the word USER 1 is displayed. Another press of the select button displays the word USER 2.

So select "your" USER memory. Then wait for 5 seconds until the display disappears.

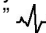
The monitor has adopted your data and is now ready for taking measurements.

Pressing the ON/OFF button starts the automatic measurement. The cuff inflates to approx. 190 mmHg pressure.

The automatic deflation starts the measurement. After about 3 seconds, a heart icon starts blinking in the LC display. If a Korotkoff's sound is detected already at that time, it means a systolic pressure of more than 150 mmHg. Then the monitor re-inflates (to about 220 mmHg).

At the end of the measurement, the result is displayed in the LCD and the cuff deflates.

## Irregular heartbeat

If your heartbeat is not regular (irregular), the intervals between pulses are different. If this happens while you are taking a measurement, the icon  appears in the LC display.

## Reasons for irregular heartbeat: (not complete)

- Sport activity, sauna, stress.
  - Measuring error such as loose cuff, wrong posture during measurement
- Cause for concern:** (not complete)
- Cardiac arrhythmia

If irregular heartbeat is displayed regularly, consult your doctor.

## Aborting a measurement

You can stop the measurement at any time by pressing the ON/OFF button. The BP monitor deflates the cuff instantly.

## The memory function

The SC 7100 saves up to 50 measurements in every memory area. Together with each measurement (systolic, diastolic and heart rate), the date and time are also saved. The measurement with the longest residence time in the memory is assigned number 1. All other measurements increment by one until number 50. Then the memory is full. The next measurement occupies memory location 50. The measurement formerly at location 50 changes to location 49 (etc., 48→47 .... 2→1). The oldest measurement formerly at location 1 is lost.

## Delete a blood pressure value from the memory

Press the MEMORY key to view the blood pressure value you want to delete. Pressing the ON/Off button instantly (and holding it for at least 4 secs.) displays dEL (for "delete") and the number of the memory location.

Press the ON/Off button instantly and the value will be deleted.

## Delete all blood pressure values in the memory

Press the MEMORY key to display any blood pressure value. Pressing the ON/Off button instantly (and holding it for at least 4 secs.) displays

## dEL (for "delete") and the number of the memory location:

Press the MEMORY key instantly and the LCD displays

## dEL (for "delete") and ALL (for all memory locations).

Press the ON/Off button instantly and all values will be deleted. Time and date are not deleted.

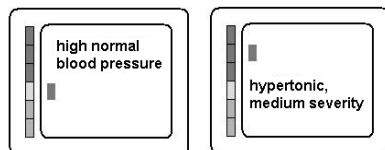
## WHO Interpretation of blood pressure readings

29/12/2007

Meaning	Systolic / mmHg	Diastolic / mmHg
optimal	< 120	< 80
normal	< 130	< 85
high-normal	130 - 139	85 - 89
mildly hypertonic	140 - 159	90 - 99
hypertonic, medium severity	160 - 179	100 - 109
severely hypertonic	≥ 180	≥ 110

The World Health Organization (WHO) recommends medical treatment, when the blood pressure value exceeds 140/90 mmHg.

## Examples:

## Cleaning and maintenance

♥ Do not expose the BP monitor to extreme temperature, moisture, dust or direct sunlight.

♥ The cuff contains a sensitive airtight bulb. Treat it with care and avoid any stress by torsion or kinking.

♥ Clean the BP monitor with a soft, dry cloth. Do not use fuel, thinner or other solvent. Remove specks on the cuff carefully with a wet cloth and soap water.

## Do not wash the cuff.

♥ Do not drop the BP monitor or treat it with force.

♥ Do not try to open the BP monitor. This will make the factory calibration invalid

## Troubleshooting and remedies

If a fault is detected during a measurement, the BM monitor displays „Err“ and a symbol (code for service personnel).

From many years of experience we know that most faults are the consequence of wrong use. The most frequent causes of faults are excessive movement during the measurement, wrong application of the cuff, use of wrong batteries or use of a rechargeable battery.

Message	Error description	Correction
LL Err	The pressure measured was lower than 20mmHg.	Please measure again.
UU Err	The pressure measured was higher than 300mmHg.	Please measure again.
P Err	Pumping error.	Please check cuff and try again.
rrErr	The pressure can not be measured due to signal noise.	Please measure again.
HI	The pumping pressure is higher than 300mmHg.	Please measure again.
 or no display	Low batteries.	Please check and replace the batteries if necessary.

Wait at least 5 minutes before repeating the measurement.

## Periodic re-calibration

We recommend periodic checks of the pressure indication every 2 years.

## Referenced norms

For registration, a defined number of blood pressure monitors was subjected to stringent clinical testing. These monitors used the same program for blood pressure measurement as your monitor uses. Thus, this program is also regarded as clinically tested.

The manufacture of the blood pressure monitor complies with the European norms for blood pressure monitors: EN 60601-1, EN 1060-1 to EN 1060-3.

## Electromagnetic compatibility:

The blood pressure monitor complies with the requirements of European norm EN 60601-1-2.

The requirements of EU directive 93/42/EEC concerning medical devices, class IIa are complied with.

## Warranty

We grant a warranty of two years for blood pressure monitor SC 7100. Within this period, defects of the BP monitor will be repaired free of charge. You can also have your BP monitor repaired by our customer service after the warranty period. We will provide you with a free cost estimate for the repair.

The warranty does not cover damage due to improper handling, accident, natural wear and tear (cuff), non-compliance with the instructions for use or unauthorized modifications of the monitor.

Excluded from the warranty are: the packaging, batteries and the instructions for use.

## Disposal note

The disposal of batteries is regulated. Spent batteries are collected at collection points provided by battery dealers and the communities.

Dispose of the blood pressure monitor as prescribed. i.e., at a collection point for electrical scrap.

## Specifications

Measuring method: Oscillometric

Measuring range:

Pressure: 20 to 280 mmHg

Heart rate: 40 to 195 / min

Measuring accuracy:

Stat. pressure: +/- 3 mmHg

Heart rate: +/- 5 %

Measuring value memory: 2 memories each with 50 locations for systolic / diastolic / heart rate, date and time

Display: LC display

Pressure pick-up: Semi-conductor pressure sensor

Automatic switch-off: after approx. 3 minutes of inactivity

Voltage supply: 2 x LR03 (alkali batteries),

Working temperature range: +10°C - +40°C

Storage temperature range: -10°C - +50°C

Cuff length: for wrist circumference from 135 mm to 195 mm

*Specifications may change as technical improvements take place*

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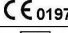
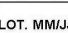
## Explanation of symbols

	<b>Manufacturer</b>
	CE-sign of a notified body (TÜV Rheinland)
	<b>Product type BF</b>
	<b>Disposal notes</b>
	<b>Read the instruction manual</b>
LOT. MM/JJ	<b>Production period (MM/YY)</b>

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SCALA Electronic GmbH
Müşteri servisi: SES GmbH